

Lab Device [Blood Segment Device](#)

LY segment device is a single-use segment sampling device makes it easy for lab technicians to complete pre-transfusion type and cross procedures while minimizing the risk of spills and breakage.

Working method

- 1 Placing the device directly on top of the test tube.
- 2 Inserting the blood segment into LY segment device.
- 3 Pressing until the needle punctures the blood segment.
- 4 Letting blood flow from the segment into the specimen tube.
- 5 Discarding the blood segment and LY segment device into medical waste container.

Attention

- 1 Wear gloves when using LY lob [blood segmen device](#).
- 2 Do not squeeze until the needle has pierced the segment.
- 3 Single-use only.
- 4 For in-vitro diagnostic purpose.

Advantages

1. Access blood from a segment safely.
2. Avoid risk of sharp injuries or splatter.
3. Reduce potential of blood sample cross contamination.



CE



FDA

www.chinalyxr.com



Key Words: Lab Device, [Blood Segment Device](#), Hematype segment alternative, disposable blood segment device, segment device for medical use