

## Ly Blood Bank Process Device

Product Name: LY New Arrival Innovation Lab Device Segment Device

LY segment device is a single-use segment sampling device makes it easy for lab technicians to complete pre-transfusion type and cross procedures while minimizing the risk of spills and breakage.

### Working method

1	Placing the device directly on top of the test tube.
2	Inserting the blood segment into LY segment device.
3	Pressing until the needle punctures the blood segment.
4	Letting blood flow from the segment into the specimen tube.
5	Discarding the blood segment and LY segment device into medical waste container.

### Attention

1	Wear gloves when using LY segmen device.
2	Do not squeeze until the needle has pierced the segment.
3	Single-use only.
4	For in-vitro diagnostic purpose.

### Advantages

1	Access blood from a segment safely.
2	Avoid risk of sharp injuries or splatter.
3	Reduce potential of blood sample cross contamination.

### Company Information

H. K Lanyuan Productive (Wuhan)Co., Limited. Factory located in Xiantao, Hubei province established in 2005. Mainly production and sales all kinds of Healthcare and Medical Disposable Products for hospitals, Clinics and daily use. With fully experience and professionally, together our painstaking efforts of these years, the products almost covering all medical accessories and operation room dressings. We never forget to introducing the high efficiency equipment, developing and improving the advanced technology in sure of the good quality products, newly products design.

Our factory covers an area of 10, 000 square meters, which has area 5, 000 square meters of production workshop, Triply-wire machine 294 sets, Flat wire machine 120 sets, Welding machine 80 sets, Mask machine 150 sets etc. Two automatic warehouse, storage capacity over 43, 000 cubic meters, With ethylene oxide sterilization center of 2, 500 square meters, sterilized 7, 500 cubic meters per month.

The average monthly production capacity of 1, 000 tons of non-woven fabrics, 20, 000PCS medical accessories, 20, 000PCS sterilized surgical drape packs, 700, 000PCS non-woven gowns. The annual output value of 20 million.

We are engaged in more than 30 categories of protective and healthcare products, they are mainly

non-woven products, plastic products and hospitable medical accessories to a significant extent.  
Please check our website for more details: <http://www.chinalyxr.com/>.

Payment, packing and delivery

Payment term L/C, T/T, Western Union

Loading Port: Shanghai, Wuhan, etc.

Delivery time About 15-20 days, based on order quantity

Image



IF YOU HAVE ANY QUESTIONS, PLEASE ALWAYS FEEL FREE TO CONTACT US. LOOKING FORWARD TO YOUR INQUIRY.