

# **Plaster of Paris Bandage for Arms and Legs Bone Fractures Fixed**

## **1.Material:**

cotton gauze and [plaster of paris](#)

## **2.Specifications:**

Bandage width 5cm, 7.5cm, 10cm, 12.5cm, 15cm, 20cm.

Normal length 2.7m, or any length according to customers' request.

<b>Specification</b>	<b>Package(rolls/ctn)</b>
5x270cm	240
7.5x270cm	240
10x270cm	120
12.5x270cm	120
15x270cm	120
20x270cm	60

## **3.Features:**

^least loss of gypsum powder when soaking by water.

^stable solidate time.

^easy to dry.

^super strength.

^easy to operate,environment friendly.

## **4.Usage:**

Suitable for bone surgery operation, orthoreadic gypsum, mask, hand crafts and arts, toys, healthy sports material for slim figure and so on.



#### 5.Attention:

- ^ Soak the bandage roll thoroughly and squeeze water outside of the water. Don't squeeze it in the water to avoid the loss of gypsum powder.
- ^ Pay attention to the evenness when tie up. Bandage roll rolls on the body, with loose side touches muscles. Don't tie up inside out, and not tear it by band to prevent annulus chain press.
- ^ Get rid of bubbles between every layers by touch until full evenness, but after the gypsum liquid becomes paste(solidifying point of gypsum), you mustn't squeeze it to avoid the break of solidified structure of the gypsum.





