

Lanyuan [blood dispenser](#): an item designed to eliminate the need to remove the stopper of a tube of blood when making a smear.

Image

Using method

- 1 Holding the tube upright, insert Lanyuan blood dispenser cannula into the lid of the tube.
- 2 Invert the tube onto a glass slide or other receptor.
- 3 Press the head of Lanyuan blood dispenser into the glass slide to produce a drop of blood.
- 4 Remove the tube and discard Lanyuan blood dispenser component or the entire tube.

Advantage

LY blood dispenser is a safe and effective tool in the lab.

- 1 A quicker blood dispensing method than applicator sticks and capillary tubes.
- 2 A simple way to get consistently sized blood drops.
- 3 An clear way to show blood flow.
- 4 An alternative to removing a tube's lid for every collection.



[www.lanyuansupply.com](http://www.lanyuansupply.com)

